



REQUIREMENT		SETTING	TO PATIENT
O ₂ % (±5%)	PEEP cmH ₂ O	O ₂ FLOW Ⓐ + Ⓑ	FLOW L/min (±8%)
40%	5	11 + 0	56
	7,5	12 + 0	58
	10	13 + 0	60
	12,5	14 + 0	66
50%	5	10 + 10	60
	7,5	11 + 10	62
	10	12 + 10	64
	12,5	13 + 10	66
60%	5	8 + 18	59
	7,5	9 + 18	61
	10	10 + 18	64
	12,5	11 + 18	65
70%	5	5 + 30	61
	7,5	6 + 30	62
	10	7 + 30	64
	12,5	8 + 30	65

Starmed Venturi valve operating principle:

by increasing A $\Rightarrow$ increased patient flow
by increasing B $\Rightarrow$ increased $O_2\%$

Always refer to the performance table for therapy settings