



InterActive™ respiratory humidifier



Service information sheet

Refer to the InterActive service manual (version) for full information.

Warranty

2 years

Maintenance Schedule	
Period	Task
Monthly	<ul style="list-style-type: none"> Visual Inspection
Year	<ul style="list-style-type: none"> Visual Inspection Electrical Safety Check Coin Cell Check Warm-up Check Temperature Accuracy Check

Installation

Ensure the following checks are performed before first use:

- Visual inspection
- Electrical safety check

***NOTE: Checks are mandatory to ensure that Servicing and Maintenance procedures have been carried out successfully and that the InterActive.**

Servicing when required

To maintain the InterActive, the service personnel will require the following equipment. Items marked with a part number can be obtained by contacting.

Complete Tools/Equipment List	
Name	Part No.
T8 Hexalobular screwdriver	N/A
Battery Removal Tool	NBL9040
Flow regulator with air supply 10 ± 2 L/min	N/A
Timer	N/A
Water-resistant calibrated thermometer accurate to $\pm 0.5^\circ\text{C}$	N/A
Container of water at $40^\circ\text{C} \pm 5^\circ\text{C}$	N/A
Intersurgical heated wire breathing system and humidification chamber	Example 2026310
InterActive Respiratory Humidifier	71000XX*
InterActive Single Heated Wire Adapter	7653000
InterActive Dual Heated Wire Adapter	7651000
InterActive Dual Temperature Probe	7661000

Spare Parts List	
Name	Part No.
InterActive software update pack	7100111
16gb microSD card 6546	
CR2032 3.3V coin cell battery	6549

*Contact Intersurgical for market specific code

InterActive serv sheet INT ▪ Issue 2 07.25



Intersurgical Ltd, Crane House, Molly Millars Lane, Wokingham, Berkshire, RG41 2RZ, UK
T: +44 (0)118 965 6300 info@intersurgical.com www.intersurgical.com



The manufacturer Intersurgical Ltd is certified to ISO 14001:2015, ISO 9001:2015, ISO 13485:2016 and MDSAP

Please think before you print

Save energy and paper. If you must print this information sheet please print it double sided.